


Introduction

Fibrinolysis plays a crucial role in maintaining an equilibrium in blood circulation. An imbalance in the fibrinolysis system can cause a variety of clinical problems such as thrombotic risk, extensive bleeding and inflammation.

The Lysis Timer is a semi-automated device for studying euglobulin lysis time in citrated human plasma. Thrombin is used to form a blood clot whilst small amounts of tPA trigger fibrinolysis. The instrument analyses the clot dissolution kinetics resulting in a clot lysis time.

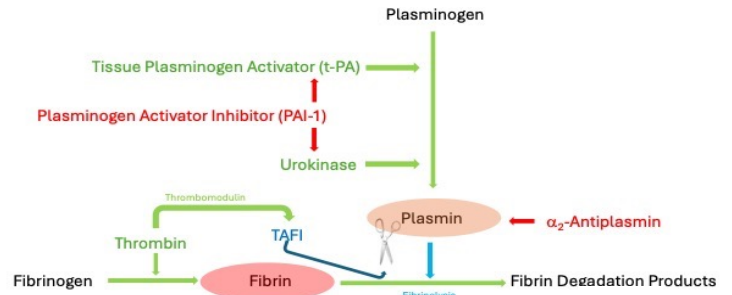


For the evaluation of the Global Fibrinolytic Capacity (GFC)




- ✓ **Standardized Test Protocol**
- ✓ **Uses only 100µl of Citrated Plasma**
- ✓ **Easy two-step semi-automated assay**
- ✓ **Sensitive to both hyper- and hypo-fibrinolysis**
- ✓ **Low, Normal and High Controls available**
- ✓ **One test provides a full fibrinolytic profile in 1 hour**
- ✓ **Strong correlation with underlying fibrinolysis parameters**
- ✓ **Sensitive to tPA related fibrinolysis deviations**

Assay principle

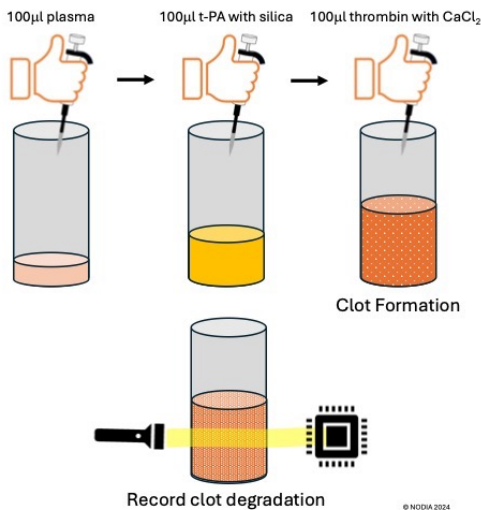

Typical results

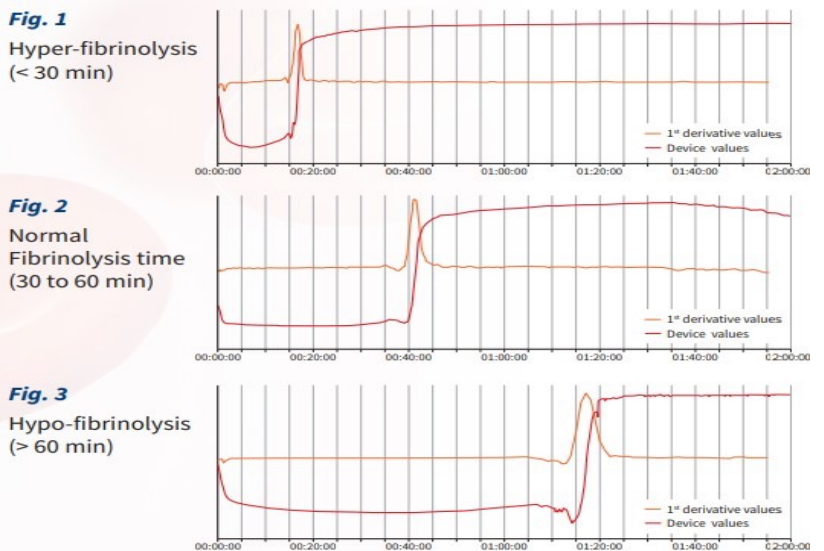

Available products

Product	Reference	Info
Lysis Timer Instrument	LT-8V	8 Channel System incl. software
Glass Tubes for Lysis Timer	LT-CV	Box of 144 tubes
GFC-Test	CK093K	Set of 3 x 2 ml
GFC Control Plasma	SC104 K	Set of 2 x 3 x 1 ml



Collect a blood sample by venipuncture using 0,109M trisodium citrate anticoagulant. Discard the first tube.

Within 2 hours, use a laboratory-validated method to obtain platelet-poor plasma. Decant the plasma in a plastic tube.


Switch on the Lysis Timer. Then run the Lysis Timer software. **The instrument will connect when 37°C is reached**


Select a measuring channel and enter patient data

Verify the S-shape of the curve
Interpret lysis time results

Indicative lysis time values:

Normal : 30-60 min.
Hyper fibrinolysis < 30 minutes
Hypo fibrinolysis > 60 minutes

Wait for the test to finish
or
Stop the reading when lysis is complete
by clicking on: 

Put the tube back in the Lysis Timer and start the reading by clicking on: 

Add 100 µL of **Reagent 2** (Thrombin reagent) to the test tube and vortex.

Take the test tube out of the instrument when the LED turns blue

1 minute incubation

Put the tube in the selected channel in the instrument
The LED will turn yellow.



Lysis Timer Quick start guide

(This guide does not replace the User Manual)



For more information please contact:

Quadratech Diagnostics Ltd
Unit 11, The Old Printworks,
20 Wharf Road,
Eastbourne, BN21 3AW
United Kingdom

T: 033 3321 2371
E: info@quadratech.co.uk
www.quadratech.co.uk

